

## AbleNet<sup>®</sup>, Inc. Repair Information

All AbleNet manufactured products are warranted for one full year from the date of purchase. If documentation identifying the date of purchase is not supplied, the warranty period will be determined as one year from the manufacture date stamped inside the product.

All products under warranty are repaired free of charge. Products no longer covered by warranty may be repaired for a nominal fee. All repairs are warranted for 90 days. Before sending in a product for repair, please contact an AbleNet customer service representative for the Return Authorization Number (RA#) and repair charge, if applicable.

**To have a product repaired please include the following with your product:**

- Name and phone number
- RA#
- Return shipping address
- Brief description of problem (what the product is or isn't doing along with what has already been done to correct the problem)
- Documentation that shows the purchase date (i.e., copy of packing slip, purchase order, or invoice)
- Check, money order, or purchase order for the amount of the repair

**Call 1-800-322-0956 for a Return Authorization Number**

**For more information about AbleNet  
products and services please call:**


**Toll Free:** 1-800-322-0956 (USA and Canada only)

**AbleNet, Inc.**  
Minneapolis • St. Paul  
Minnesota

## User Manual

---

# Battery Device Adapter

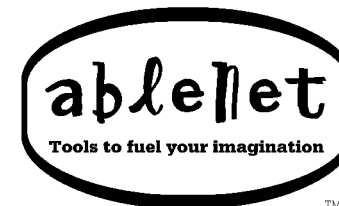

## To Use the Battery Device Adapter

1. Place the copper disk between the battery and the battery contact in the toy. Close the battery compartment cover. You may need to remove a portion of the cover to allow the wire to fit through. Use a file to make a small notch in the battery compartment cover.
2. Plug a switch into the Battery Device Adapter.
3. Turn the toy's on/off control to the "on" position. Now the switch can be used to turn the toy on or off.

## Equipment Trouble?

If you followed the instructions and the device fails to operate, check the following:

1. Does the battery-operated toy work independently if the Battery Device Adapter is removed. If not, a problem may exist with the toy or device. Check the batteries in another toy to ensure the batteries work.
2. Does the switch work with other battery-operated toys or control units? If so, a problem may exist with the Battery Device Adapter.

## Still Not Working?

If the AbleNet® Battery Device Adapter still refuses to work, it may be defective. DO NOT ATTEMPT TO TAKE IT APART. Call an AbleNet Customer Service representative at (800) 322-0956 for assistance.

## Warranty

AbleNet offers a one-year limited warranty on all AbleNet products; other manufacturer's products carry manufacturer's warranty. Warranty in full text is available upon request.

## Guarantee

AbleNet offers a 30-day money back guarantee if our products do not meet your needs or expectations. Products must be in new condition to honor this service. **Please follow our repair/return policy when sending products back under our guarantee.**