

# Contents

<b>Description .....</b>	<b>2</b>
<b>Switch Control .....</b>	<b>2</b>
<b>Operating Mode .....</b>	<b>2</b>
<b>Training the Mini Relax.....</b>	<b>2</b>
<b>Use Mode .....</b>	<b>3</b>
<b>Battery Replacement .....</b>	<b>3</b>

## **Description**

The Mini Relax with Jacks is a portable infrared transmitter which can be accessed using a single switch or up to 6 single switches. The Mini Relax *learns* codes or signals from any regular infrared remote control that is used for TV's, VCR's, etc. It is capable of learning, and transmitting six different signals from a variety of regular appliance transmitters.

## **Switch Control**

Up to 6 single switches are required to operate the Mini Relax. Plug the switches into the jacks on the right-hand side of the unit.

## **Operating Mode**

The Mini Relax has two operating modes: one is TRAIN, in which it *learns* the required signal, and the other is USE, where it transmits the *learned* signal. The mode is selected by the switch on the side of the unit.

## **Training the Mini Relax**

The Mini Relax may be trained to reproduce up to six signals or functions from a variety of infrared transmitters. Each signal may be independently retrained at any time.

1. Put the Mini Relax into the TRAIN mode.
2. Place the Mini Relax and the regular remote control nose to nose on a flat surface; that is, the transmitting window of the regular remote control must be facing the top window of the Mini Relax.

3. Select an appropriate location on the Mini Relax display for the signal you wish to store. Do this by activating the switch where you wish the signal to be stored. (You may terminate TRAIN mode at any time by switching back to USE mode. This will not affect signals previously stored).
  4. Once at the desired LED/location for training, press and hold the button of the desired function on the regular remote control. After a short delay, the Mini Relax will emit a long beep and the LED will go out. The Mini Relax has now learned that specific function.
  5. Repeat steps 3 and 4 for other locations to be trained.
- \* When training, it may take up to 5 seconds.
  - \* Some of the new remotes send two signals. The first is a short burst followed by a second signal.
  - \* If your remote is not training, hold for a longer time in case your remote is sending two signals.

## **Use Mode**

1. Ensure that you are in USE mode.
2. Plug a single switch into the corresponding jack located on the side of the unit.
3. Press the corresponding switch to send the infrared transmission

## **Battery Replacement**

The battery compartment can be found on the underside of the unit. Simply slide the black cover in the direction of the arrow to remove. Attach a 9 volt alkaline battery to the battery clip. Replace cover.

**ANY QUESTIONS? Contact your local supplier or TASH:**




*Unit 1, 91 Station Street  
Ajax, Ontario  
L1S 3H2  
Canada*

*Tel: (905) 686-4129 Toll-free: 1-800-463-5685  
Fax: (905) 686-6895 e-mail: [tashinc@aol.com](mailto:tashinc@aol.com)*

*Website address: [www.tashinc.com](http://www.tashinc.com)*

# Mini Relax with Jacks

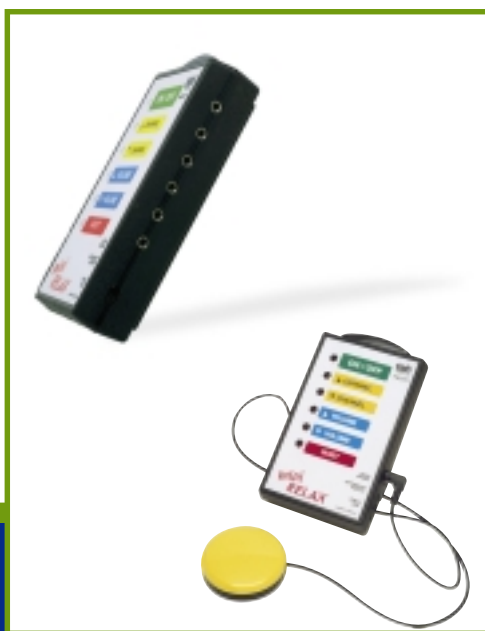

Real People – Real Solutions

[www.tashinc.com](http://www.tashinc.com) • 1-800-463-5685